



Gelclair is a viscous oral gel for the management and relief of pain associated with oral lesions, particularly oral mucositis/stomatitis, which may be caused by chemotherapy or radiation therapy and irritation from oral surgery. Gelclair forms a protective coating over the oral mucosa which shields exposed or sensitized nerve endings from over-stimulation and provides oral pain relief. It does not irritate or sting and is non-numbing.

Contents: Each sachet contains 15 mL.

Directions for Use:

Initiate treatment with Gelclair as soon as signs and symptoms of mucositis develop, in order to provide the maximum benefit.

- Pour the entire contents of the single-dose Gelclair sachet into a glass and add up to 40 mL or 3 tablespoons of water. If water is not available, Gelclair may be used undiluted.
- Stir mixture and use at once.
- Rinse around mouth for at least one minute or as long as possible to coat tongue, palate, throat, inside of cheeks and all oral tissue thoroughly.
- Gargle and spit out - do not swallow. If accidentally swallowed, no side-effects are anticipated.
- Discard any unused mouthwash.
- Use at least 3 times per day or as needed.
- Do not eat or drink for at least one hour following treatment.
- If performing oral care (e.g. brushing teeth), it should be undertaken prior to using Gelclair.

Keep out of reach of children.

Store below 25°C. Do not refrigerate.

Contains:

Purified water, Maltodextrin, Propylene Glycol, Polyvinylpyrrolidone, Sodium Hyaluronate, Potassium Sorbate, Sodium Benzoate, Hydroxyethylcellulose, PEG-40 Hydrogenated Castor Oil, Disodium Edetate, Benzalkonium Chloride, Flavouring, Saccharin Sodium, Glycyrrhetic Acid



Biokosmes Srl
Lecco, Italy

GELCLAIR® is a trademark of Helsinn Healthcare SA, Switzerland.

Gelclair is distributed by Orphan Australia Pty. Ltd., 48 Kangan Drive Berwick Victoria 3806 Australia under licence from Helsinn Healthcare SA, Switzerland.

Tel: 03 9769 5744

Fax: 03 9769 5944

Web: www.orphan.com.au

APO32837/1
Gelclairleaflet/0906